



TrabEx+™





TrabEx+™ Handpiece

IFU

Goniolens






Goniolens "L"


Goniolens "R"

Irrigating Goniectomy™ A New Angle on Glaucoma Management

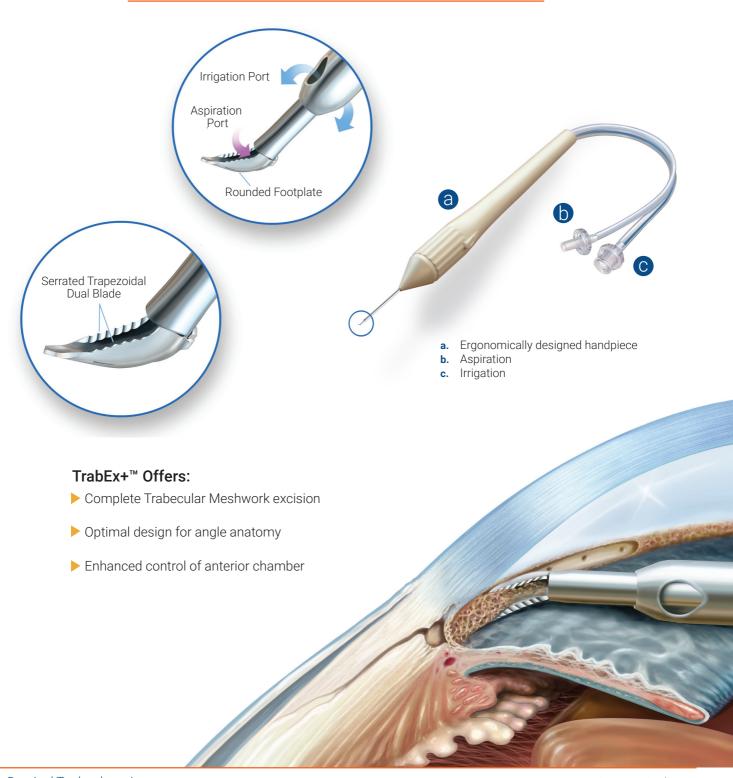


Assisted with Irrigation & Aspiration

TrabEx+ with Phaco I/A Source

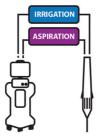


Irrigation: > 80 cm above patient head Aspiration: 10 cc/min vac = 100 - 150 mmHg Tip Entering Trabecular Meshwork (TM)

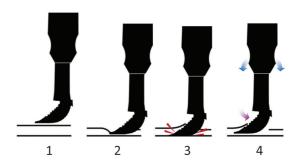

- ► TrabEx+™ with I/A can be used to perform a stand alone procedure or combined with cataract extraction
- ▶ I/A maintains deep, stable, and clear anterior chamber & improves angle visualization
- ▶ Rounded footplate guides tip along contour of Schlemm's Canal
- ▶ Precise laser-honed, serrated blades facilitate complete TM removal
- ▶ Viscolelastic is not needed
- ▶ Patent: www.microsurgical.com/patents



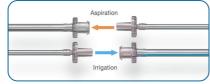





VAC 100 mmHg



TrabEx+™

Product Number

600021-02 TrabEx+ with I/A Handpiece

600009 Goniolens "L" (for left-handed surgeon), re-usable

Goniolens "R" (for right-handed surgeon), re-usable

CPT Code

65820: Goniotomy

Order Online

www.microsurgical.com

To reach your local sales representative contact customer service:

U.S.: customerservice@microsurgical.com International: MSTinternationalCS@microsurgical.com

1-888-279-3323

MicroSurgical Technology 8415 154th Ave NE Redmond, WA 98052

